


MADE IN ITALY

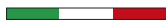

Hyperoil®

WWW.HYPEROIL.COM

ADVANCED SKIN & WOUND MANAGEMENT

Helps regeneration and skin tissue repair

HERBAL BASED



Class IIb Medical Device

GEL FORMULATION



5 ml Gel Re-Sealable Vial
(individually packed)
Ref. 100501HY



30 ml Gel Tube
(with applicator)
Ref. 10030TGO



100 ml Gel Dispenser
(with spray nozzle)
Ref. 100100SPO



10 pcs (10x10cm)
Medicated Gauzes
(individually packed)
Ref. 10010GMOS

OIL FORMULATION



5 ml Oil Re-Sealable Vial
(individually packed)
Ref. 100101HY



50 ml Glass Dropping Bottle
(with dropper)
Ref. 10050CGO



100 ml Glass Bottle
(with spray nozzle)
Ref. 100610

A SYNERGY OF PLANTS PROPERTIES

The combined overall effect of the two plants contained in **HYPEROIL®** is greater than the sum of their individual effects (bibliography available upon request).



NEEM

(Azadirachtin)

- Antiseptic
- Antimicrobial
- Antibacterial
- Antiviral
- Anti-inflammatory
- Antifungal
- Healing and Regenerating

HYPERICUM

(Hyperforin)

- Tissue regeneration
- Re-epithelialization
- Anti-inflammatory
- Antiseptic Soothing





EASY TO USE: HEALTHCARE PROFESSIONALS AND CAREGIVERS HAVE AN EASY SOLUTION AMONG THE COUNTLESS OPTIONS AVAILABLE ON THE MARKET.

IN-HOME WOUND CARE

PATIENT AUTONOMY
PREVENTS AMPUTATION



TRAUMATIC INJURY IN DIALYSIS DIABETIC

HYPEROIL® OIL every day, till day 77.
Followed by HYPEROIL® OIL
twice a week until complete recovery.



Day 0



Day 77



Day 84

SKIN REGENERATION AND ELASTICITY



ABRASION

Acute phase: HYPEROIL® OIL twice a day.
Maturation phase: HYPEROIL® GEL twice a day.



Day 0



Day 35

FASTER WOUND HEALING



BURN

HYPEROIL® OIL once a day.



Day 0



Day 7

WWW.HYPEROIL.COM

THANKS TO THE HERBAL-BASED / MULTI-FUNCTIONAL COMPOSITION, HYPEROIL® CARRIES NO RISK ASSOCIATED WITH OVERDOSAGE, PAIN OR TRAUMA AT DRESSING CHANGE.



Day 0



Day 44



**SELF-MANAGEMENT
EMPOWERED LIFE
QUALITY**

SURGICAL WOUND DEHISCENCE (Abdominoplasty surgery outcome)

HYPEROIL® GEL every day for 15 days then
HYPEROIL® GEL twice a week until healing.



Day 0



Day 42



**PREVENTION AND
MANAGEMENT OF
RADIATION-INDUCED SKIN
INJURIES**

RADIATION DERMATITIS

HYPEROIL® OIL two hours before radiotherapy
and immediately after.



Day 4



Day 124



COST-EFFECTIVENESS

DECUBITUS ULCER

HYPEROIL® OIL once a day.

FORMULATIONS

HYPEROIL® is available in both GEL and OIL formulation to treat a variety of skin injuries.

Active ingredients are more diluted in the **GEL** formulation with no impact on its efficacy.

Higher concentration of active ingredients in **OIL** formulation helps to keep infections under control, thus enhancing a cleansing/debridement action.

Oil is also recommended when thick gel structure is unable to reach the treatment area.

HYPEROIL® is a topical ointment that promotes wound healing for external wounds.

HOW MUCH HYPEROIL® TO APPLY?

Quantity may vary according to the wound you are managing (size, depth).

Always apply a thin layer of Hyperoil® GEL or OIL to form a thin, invisible barrier.

Remember to go over the edges of the wound, to ensure proper tissue regeneration and restoration. Do not exceed in the amount.

FOR ALL KIND OF WOUNDS

Acute - Chronic - Infected

One solution for every phase of wound healing



SCIENTIFIC LITERATURE

Buruli Ulcer

- Iabichella ML et al.(2014), Localized Treatment of Chronic Buruli Ulcer with Hyperoil™: An Unexpected Outcome, *Austin J Clin Case Rep*, 1(7): 3
- Iabichella ML et al. (2015), Ulcère de Buruli: des Horizons Thérapeutiques en Hôpital et en Brousse, *Angéiologie*, 67(1): 29-41

Diabetic Foot

- Iabichella ML (2013), The Use of an Extract of *Hypericum perforatum* and *Azadirachta indica* in Advanced Diabetic Foot: an Unexpected Outcome, *BMJ Case Reports*, doi: 10.1136/bcr-2012-007299
- Iabichella ML, Caruso C, Lugli M (2014), The Use of an Extract of *Hypericum perforatum* and *Azadirachta indica* in a Neuropathic Patient with Advanced Diabetic Foot, *BMJ Case Reports*, doi: 10.1136/bcr-2014-205706
- Iabichella ML (2015), The Use of a Mixture of *Hypericum perforatum* and *Azadirachta indica* for the Management of Diabetic Foot Ulcers: a Case Series, *J Diabetes Metab*, 6:2, doi: 10.4172/2155-6156.1000499

Radiation Dermatitis

- Franco et al. (2014), *Hypericum perforatum* and Neem Oil for the Management of Acute Skin Toxicity in Head and Neck Cancer Patients Undergoing Radiation or Chemo-radiation: a Single-arm Prospective Observational Study, *Radiation Oncology*, 9:297, doi: 10.1186/s13014-014-0297-0

Severe Acute Malnutrition


- Iabichella ML et al. (2017), A Case of Skin Ulcer in Severe Acute Malnutrition Treated with Hyperoil®, *Austin J Clin Case Rep*, 4(1): 1111

Systemic Sclerosis

- Giuggioli D. et al. (2020) Use of Neem oil and *Hypericum Perforatum* for treatment of calcinosis-related skin ulcers in systemic sclerosis. *J Int Med Res*, 48(4):30060519882176.

WWW.HYPEROIL.COM

MADE IN ITALY

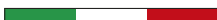

FIND OUT MORE:



FOR PROFESSIONAL USE ONLY

Hyperoil® is a product of:



RI.MOS.
*Disposable
Medical Products*



via Manuzio, 15
41037 Mirandola (MO) - Italy
Ph +39 0535 25755
Fax +39 0535 25509
info@rimos.com
www.rimos.com

Certified Company



ISO 13485

UNI CEI EN ISO
13485:2016

